Production

JUVORA™ dental disc blanks can only be processed on suitable milling machines with appropriate milling programs. The milling head must be suitable for machining PEEK polymer. Juvora Ltd recommends suitable adaptation of the CAM software by the software manufacturer to allow milling of the JUVORA™ dental disc. For an exact fit, a 5-axis milling machine is recommended. Juvora Ltd also recommends cooling the milling head for more accurate milling and to prolong its useful life.*See footnote.

Prosthesis removal from the JUVORA™ dental disc

Remove the prosthesis carefully from the blank using a hard metal cutter, and pay particular attention to the edges of the prosthesis.

Safety Information

While machining the JUVORA™ dental disc, the following safety precautions are recommended:

- Dust mask or dust extraction
- Personal protective equipment (eye protection, gloves)

The EUH210 and EUH212 warning statements present on label relate to EU Regulation (EC) No 1272/2008 only.

Storage Information

The JUVORATM dental disc should be stored in dry storage and avoid exposure to direct sunlight. The PEEK polymer from which the JUVORATM dental disc is made, is stable and can be stored for an extended period of time (10 year shelf life). It has a working temperature range from cryogenic up to 250° C and hence the storage temperature range for the JUVORATM dental disc is any ambient temperature and humidity.

Additional Information

For additional information contact info@invibio.com

*Note: Talk to the manufacturer of your machine about the processing of PEEK. Follow the instructions of your CAM and the manufacturer of your machine relating to the processing of JUVORA dental discs.

Juvora

Instructions for use

Device: JUVORA[™] Dental Disc

Caution: US Federal Law restricts this device to sale by or on the order of a dentist

Juvora Ltd, Technology Centre, Hillhouse International, Thornton Cleveleys, Lancashire, FY5 4QD, UK

Tel: +44 (0)1253 898000 · www.invibio.com

IDCM-JUV-IFU-0001, Rev

Indications

- For the manufacture of:
 - i) full and partial removable dentures and implant overdentures.
 - ii) copings, substructures (cemented or uncemented), frameworks for permanent and transitional anterior or posterior crowns and bridgework.

Contraindications

Not to be used in bridges with 2 or more pontics.

Precautions

To be manufactured with material thickness/size in accordance with the following:

- The minimum thickness of secondary crowns and matrix housings should not be less than 0.7 mm
- The minimum thickness for "conventional" retention grids should not be less than 2 mm
- Minimum size of horizontal retainer in lower jaw 2 mm x 8 mm
- Minimum size of horizontal band in upper jaw 2 mm x 8 m

Warning:

- Do not mill the frameworks too thin. Follow the milling parameter guidance provided in the Juvora Processing Guide
- Do not repair or extend frameworks after milling or initial implantation

Material Information

JUVORA™ dental discs are manufactured from Invibio PEEK polymer, a Type 3 thermoplastic according to ISO 20795 – Denture Base Polymer.

Form: Solid
Odour: Odourless
pH (Value): Not known
Boiling Point (°C): Not known
Melting Point (°C): 343

Flash Point (°C): Not known

Auto Ignition Temperature (°C): 595

Oxidising Properties:

Vapour Pressure (Pascal):

Solubility (Water):

Not applicable

Not known

Insoluble

Processing Instructions

Detailed processing instructions are provided and must be observed by the end users (dental labs and technicians) when milling frameworks from the JUVORA™ dental disc. Dental labs that are supplied with the disc will be certified through Juvora Ltd, a process that involves assessment of their capability to follow the processing guidance to manufacture high quality frameworks and finished dentures where applicable using their existing equipment. This assessment will be completed by Juvora Ltd or their certified milling centres.